

USE INSTRUCTIONS

- ▶When preparing the packages leave a 2 cm space between the sealing areas;
- ▶Protect the pointy/sharp instruments with gauze or cotton so they do not damage the packages;
- ▶Use a sealer which provides a sealing area of at least 6 mm wide;
- ▶Standardize the packages;
- ▶White the sterilization date on the envelope with a proper pen;
- ▶After autoclaving, the pink indicator must turn brown;
- ▶For a better cleaning and care of your instruments, use enzymatic detergent and an ultrasonic cleaner;
- ▶Test your autoclave with biological indicators every 7 days.

Storage Conditions

Store in a dry and ventilated place;
Keep away from sunlight;

Temperature: Between 18 and 25°C - **Humidity:** Between 30 and 60%

In case of doubt, contact us through the e-mail:
consultoria@crisofoli.com

Cristófoli Biosafety Consultant - Liliana Junqueira de P. Donatelli
Biologist CRB 18469/01-D

**Manufacture
Date**



**Single Use Product.
Reuse Forbidden.**

Shelf life: 2 years from the manufacture date.

Product composition:

Surgical grade paper, polyethylene/polypropylene, synthetic resin and ink (non toxic materials).